

# ACTIVE WHITENING

## TEETH WHITENING KIT

USER MANUAL

**INGREDIENTS:** Carbamide Peroxide (35%), glycerol, propylene glycol, Polyvinyl Pyrrolidone, Xanathan gum.

**USAGE:** Teeth stains from coffee, tabaco, food colouring, aging and yellow teeth.


**STORAGE:** Keep in cool area, away from sunlight. Discard after 12 months.

**WARNINGS:** Keep out of reach of children. Product is not intended for use for children under the age of 12. Avoid ingestion and contact with eyes and skin. If contact occurs, wipe gel off and rinse with water. Do not swallow components. Active Whitening Kit is not a substitute for regular dental care. DO NOT use if pregnant or lactating, or if teeth and gums are not healthy.

## INSTRUCTIONS


### 1 REMOVE CAP

Remove the cap on the gel syringe and replace with the new syringe tip in our box.

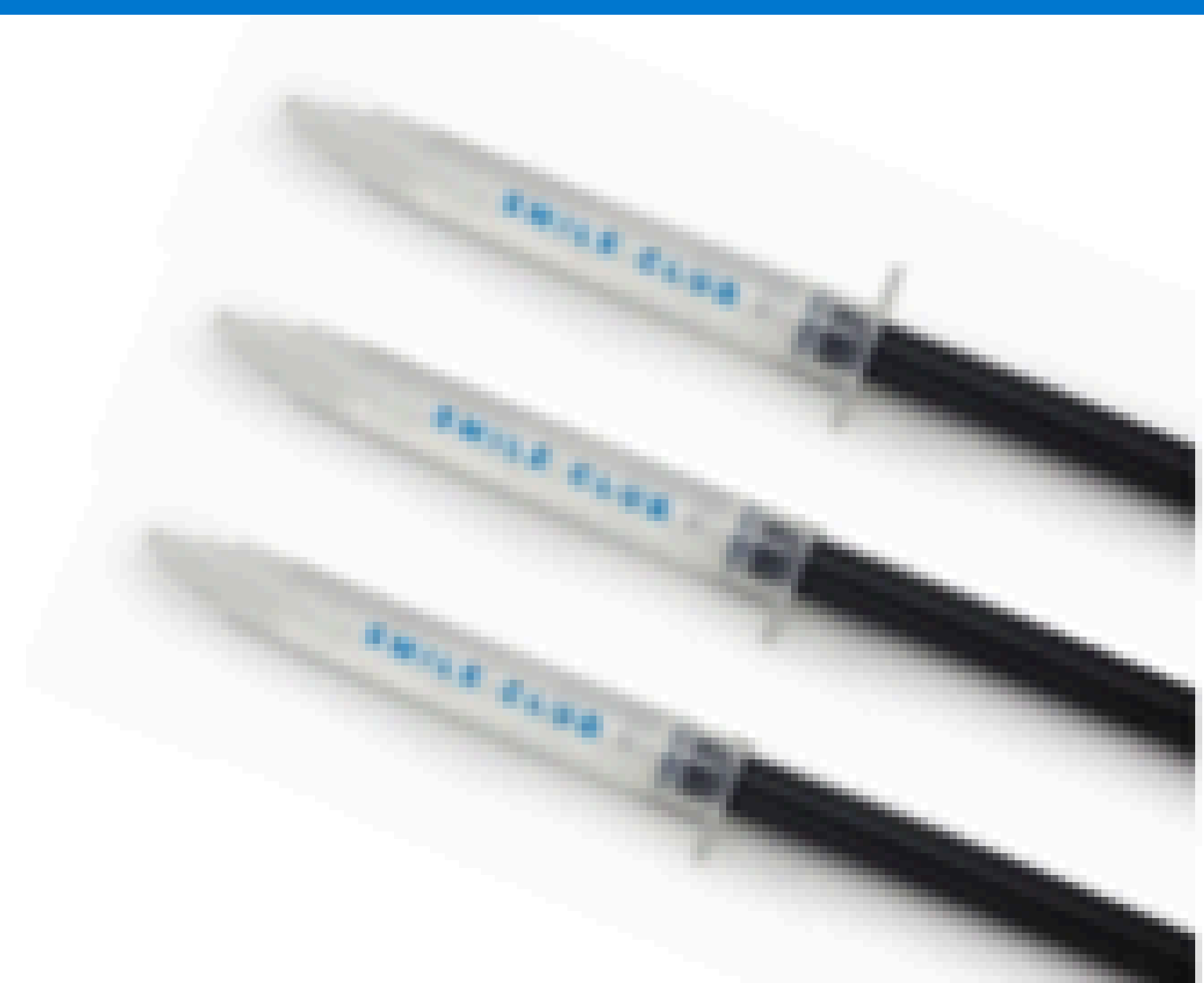

### 2 APPLICATION

Place a small drop of gel on each tooth impression on the upper and lower mouth tray.  
NB: DO NOT OVERFILL.  
LESS IS BEST.

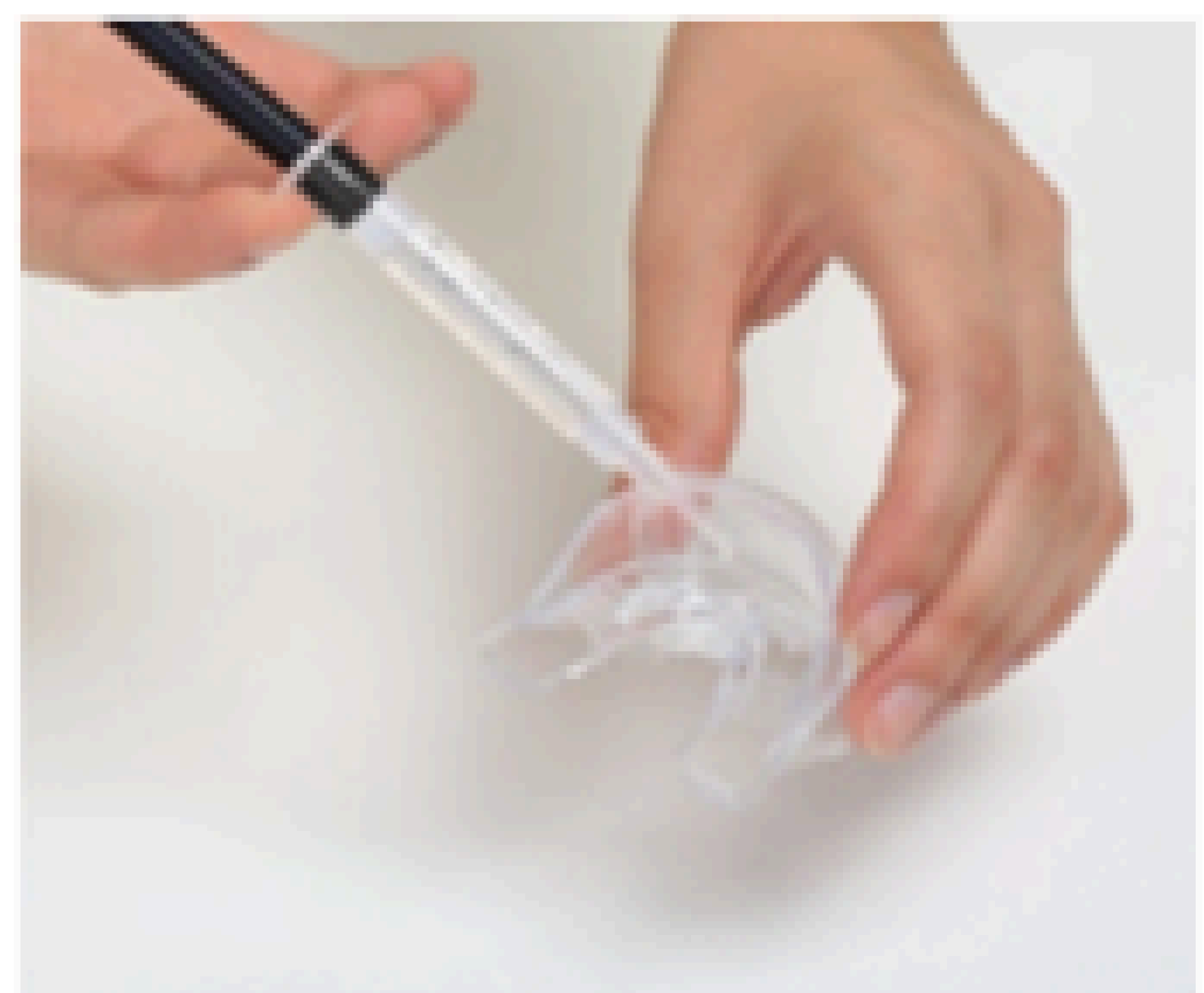

# Whiten your teeth up to 5 shades in only 5 days!

3

## WHITEN


**R**emove battery plastic protector in the mini light. Attach the duplex mouth tray to the mini light. Wear the mouth tray for approximately 30 - 45 minutes total time.



4

## RINSE AND CLEAN

Remove the mouth tray and rinse with lukewarm water. Clean whitening tray with cold water, toothpaste and toothbrush only. If old whitening gel is difficult to remove, use cotton buds to assist with removing. Thoroughly rinse with cold water and dry.

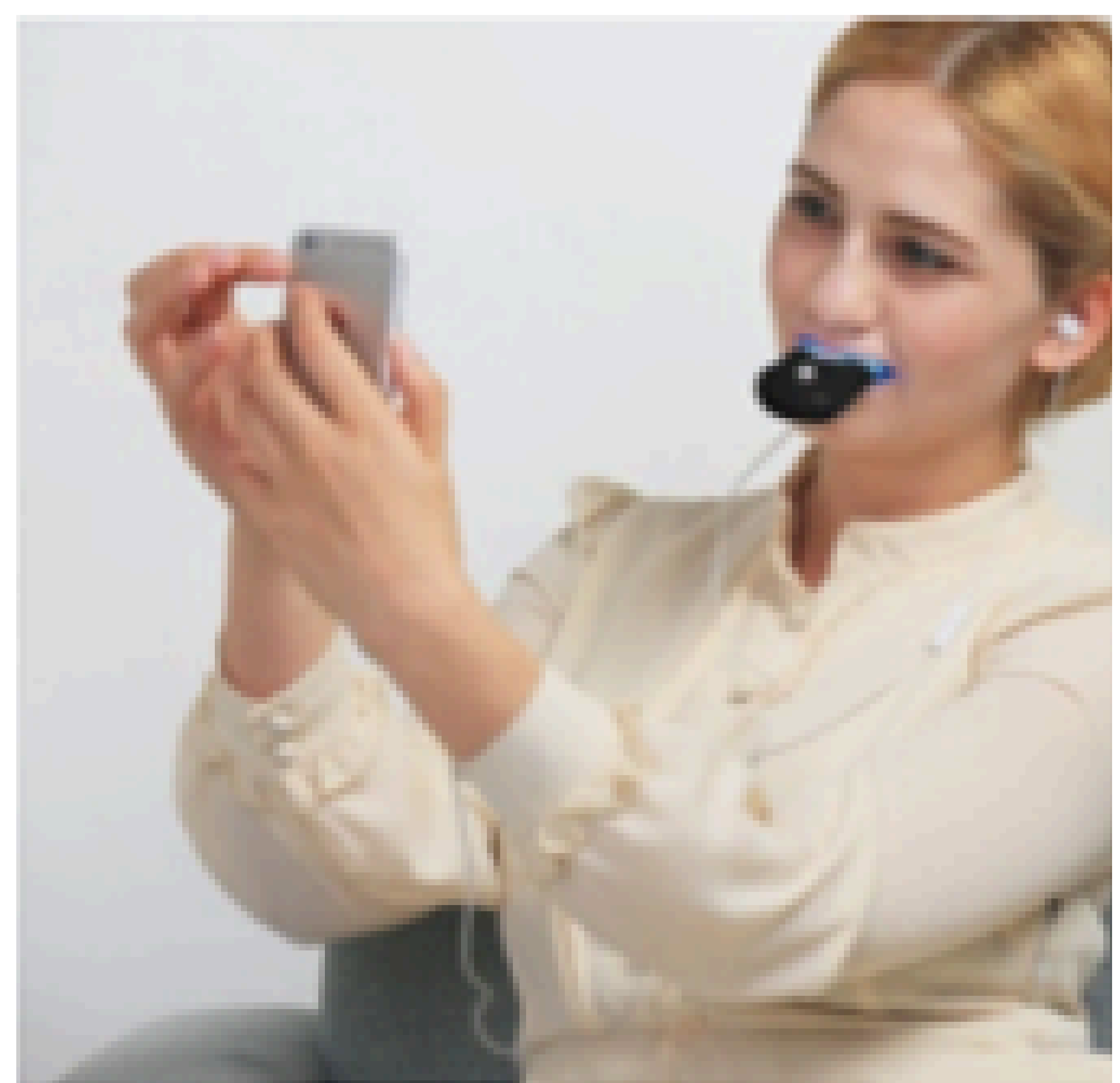

**TIPS:** Light must not be cleaned with water.

Please clean the mini LED light with paper towel or cotton swab.



# TRACK YOUR PROGRESS

